

MAGic EyeMerge Hot Keys

The following hot keys use the **F12** modifier, which is the application's default modifier key. You must press the modifier key in combination with the hot key to activate the command. If you change the default modifier key, substitute the modifier key that you are using when issuing a command. Refer to the MAGic EyeMerge help for more information about the modifier key.

Zoom and Focus

Description	Modifier Key+Hot Key
Increase zoom	F12+F2
Decrease zoom	F12+F1
Find – zoom out with crosshairs; press again to remove crosshairs and return to previous zoom level	F12+A
Focus lock – turn Auto Focus on or off (lock)	F12+D
Freeze Frame toggle	F12+Z

Screen View

Screen View Description	Modifier Key+Hot Key
Select one of three views: Full Screen – Camera view, Full Screen – Computer view, or Split Screen view	F12+V (add SHIFT to reverse order)
Full Screen – Camera view	F12+HOME
Full Screen – Computer view	F12+END
Split Screen view	F12+PAGE UP
Split Screen orientation – toggle Split Screen view between horizontal and vertical orientation; use this keystroke in conjunction with F12+F to change camera view locations	F12+P
Move Split Screen view – toggle the camera position to move the Split Screen view from top to bottom or from left to right based on screen orientation (F12+P)	F12+F

Screen View Description	Modifier Key+Hot Key
Increase Split Screen viewing area	F12+F5
Decrease Split Screen viewing area	F12+F6
Camera flip – switch between Normal, Mirror, Upside Down, or Mirror Upside Down views	F12+Q (add SHIFT to reverse order)
Move EyeMerge console to foreground	F12+ENTER
Scroll image up or left while in Split Screen mode	F12+F8
Scroll image down or right while in Split Screen mode	F12+F7

Video Modes

Video Modes Description	Modifier Key+Hot Key
Select Color Video mode – switch between Normal color mode, Negative (inverted color) mode, Black on White (positive) mode, or White on Black (negative) mode	F12+E (add SHIFT to reverse order)
Normal (full color) mode	F12+SHIFT+A
Negative (inverted color) mode	F12+SHIFT+S
Black on white (positive) mode	F12+SHIFT+D
White on black (negative) mode	F12+SHIFT+F

Capture Image

Capture Image Description	Modifier Key+Hot Key
Save current screen image	F12+CTRL+S
Capture image	F12+TAB
Delete image	F12+DELETE

View Image

View Image Description	Modifier Key+Hot Key
Jump to live image	F12+0 (zero)
Next image	F12+SPACEBAR
Previous image	F12+SHIFT+SPACEBAR
Jump to image 1	F12+1
Jump to image 2	F12+2

View Image Description	Modifier Key+Hot Key
Jump to image 3	F12+3
Jump to image 4	F12+4
Jump to image 5	F12+5
Jump to image 6	F12+6
Jump to image 7	F12+7
Jump to image 8	F12+8
Jump to image 9	F12+9
Open image file manager	F12+G

Lines and Masks

Lines and Masks Description	Modifier Key+Hot Key
Screen markers – toggle on or off	F12+K
Screen marker style – select reading lines or screen masking	F12+Y
Screen marker orientation – toggle between horizontal or vertical screen marker orientation	F12+O
Move reading lines or screen mask down or to the right	F12+N
Move reading lines or screen mask up or to the left	F12+SHIFT+N
Increase distance between reading lines or increase amount of image viewable in the masked area	F12+F10

Lines and Masks Description	Modifier Key+Hot Key
Decrease distance between reading lines or decrease amount of image viewable in the masked area	F12+F9
Select reading line color	F12+R (add SHIFT to reverse order)

Camera Control

Camera Control Description	Modifier Key+Hot Key
Camera power on or off	F12+X
Refresh video view	F12+ESCAPE
Change the camera source	F12+S

Picture Adjustment

Picture Adjustment Description	Modifier Key+Hot Key
Increase brightness	F12+B
Decrease brightness	F12+SHIFT+B
Increase hue	F12+H
Decrease hue	F12+SHIFT+H
Increase saturation	F12+U
Decrease saturation	F12+SHIFT+U
Increase contrast	F12+C
Decrease contrast	F12+SHIFT+C